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Always test your drugs. Carry Naloxone. Never use alone.

Drug Testing Strips Instructions

QUICK REFERENCE. FOR FULL INSTRUCTIONS VISIT CHANGE ON HAMILTON OR SCAN QR CODE

IMPORTANT!

When fentanyl and xylazine are added to drugs, it's not evenly mixed. Rather it's like a chocolate chip cookie where the chips are fentanyl or xylazine.

You may test from a spot that contains no fentanyl or

xylazine. It's best to test your entire supply and evaporate the water to get your drugs back.

However, as a backup method, you can make sure you crush up any rocks or crystals then thoroughly shake/or stir the baggie before you sample the supply.

PLEASE NOTE:

The strip does not test the purity or amount of fentanyl or xylazine in your drugs. The test does not indicate any drug is safe to use. Errors in testing procedure and other factors may influence the results giving false positives or negatives. The tests are given for harm reduction purposes.

YOU WILL NEED

Timer

Drugs


1 Teaspoon (5 ml)


10 milligram scoop

Drug testing strip


1. Mix by crushing & shaking.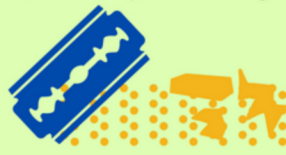


2. Measure & dilute.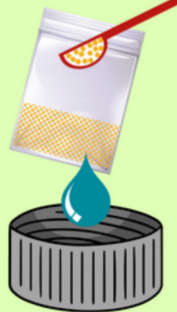


1 flat microscop = 10 milligrams.

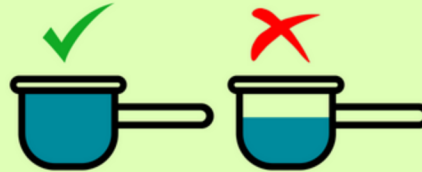1 teaspoon of clean water.

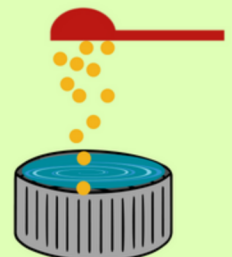3. Stir & Test.

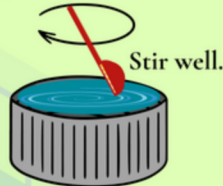Fentanyl/Xylazine

Dip wavy lines into sample. Do not go past blue line.

Lay flat. Wait 5 min.

T C

T C

No Fentanyl/Xylazine

KEEP YOUR RED MICROSCOOP AND YOUR MEASURING SPOON. FOR ADDITIONAL TEST STRIPS, VISIT CHANGE ON HAMILTON.

927 Hamilton, Allentown (484) 350-3916 M-F from 9am- 4:30pm

Made possible by Lehigh County Drug & Alcohol